
SAFETY INSTRUCTIONS

ELECTRIC WEARABLE BREAST PUMP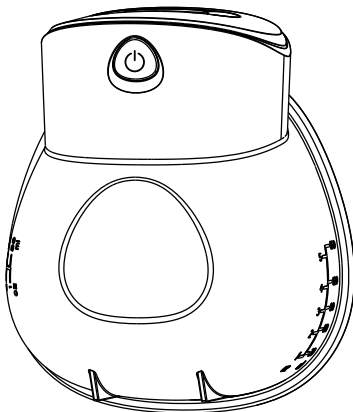

For more information,
scan the QR code to see
the extended manual.

EN	Safety instructions	3
DE	Sicherheitshinweise	11
FR	Consignes de sécurité	20
NL	Veiligheidsvoorschriften	30
IT	Istruzioni di sicurezza	39
ES	Instrucciones de seguridad	48
PT	Instruções de segurança	57
SV	Säkerhetsanvisningar	66
FI	Turvallisuusohjeet	74
NO	Sikkerhetsinstruksjoner	82
DA	Sikkerhedsinstruktioner	90
HU	Biztonsági utasítások	98
PL	Instrukcje bezpieczeństwa	107
EL	Οδηγίες ασφάλειας	117
SK	Bezpečnostné pokyny	126
CS	Bezpečnostní pokyny	135
RO	Instrucțiuni de siguranță	143

ELECTRIC WEARABLE BREAST PUMP

BCB50

For more information see the extended manual online:
alectohome.nl

About this document

This document is the user manual and contains all the information for correct, efficient, and safe use of the product.

Ensure you have fully read and understood the instructions in this document before you install or use the product. Keep the packaging and this document for future reference.

Symbols used

⚠ WARNING!	Signal word used to indicate a potentially hazardous situation which, if not avoided, could result in death or serious injury.
⚠ CAUTION!	Signal word used to indicate a potentially hazardous situation which, if not avoided, could result in minor or moderate injury.
NOTICE	Signal word used to address practices not related to physical injury.

Specifications

Input power	AC 100 - 240 V ~50/60 Hz
Output power	DC 5 V - 1 A
IP rating	IP22 Protected against solid foreign objects of 12.5 mm Ø and greater. Protected against vertically falling water drops when enclosure tilted up to 15°.

Intended use

This product is an electric wearable breast pump intended to express and collect milk from lactating breasts.

The product is intended for a single user. Never share the product with other persons.

Never use the product on body parts other than the breast.

This product is intended to be used in household and similar applications.

Only use the product as described in this user manual.

Any modification of the product may have consequences for safety, warranty and proper functioning.

Safety instructions

Ensure you have fully read and understood the instructions in this document before you install or use the product. Keep the packaging and this document for future reference.

⚠ WARNING!

Health hazard:

- Only use the product as described in this document.
- This appliance is not intended for use by persons (including children up to 14 years) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Do not let children unattended when the product is in use to ensure that they do not play with the product. Always supervise children when the product is in use.
- Cleaning and user maintenance shall not be done by children without supervision.
- Risk of pacemaker malfunction due to electromagnetic interference (EMI). Never use the product if you have a pacemaker or other implanted electronic medical device unless approved by your physician.
- Risk of inducing labor. Never use the product while pregnant. If you become pregnant while breastfeeding or breast pumping, consult with a licensed healthcare professional before continuing using the product.
- Risk of virus transmission to the baby through breast milk. Never use the product if you are infected with Hepatitis B, Hepatitis C, or HIV unless approved by your physician.
- Risk of milk cross-contamination. This is a single-user product. Never share the product with other users. Use by more than one person may present a health risk and voids the warranty.

- Risk of milk contamination or reduced product functionality. Always visually inspect all parts on damage and discoloration before each use. Do not use the product if a part is damaged or defective. Always replace a damaged or defective product immediately.
- Risk of milk contamination due to incorrect cleaning. Always clean all parts that come into contact with your breast or breast milk before first use and after every use. Always sanitise all parts except the motor before first use and once a day after use. Clean and sanitise the pump parts correctly:
 - Never place parts directly in the sink to rinse or wash. Always use a clean wash basin used only for infant items.
 - Always use drinking-quality water or bottled water to clean the parts.
 - Never use disinfectants for cleaning. Use a mild dish washing liquid.
 - During sanitising, boil the parts at least 10 minutes.
 - Store the dry parts in a clean bag/container until next use. Never store wet parts in an airtight container.
- Risk of milk contamination due to incorrect use. Always wash your hands thoroughly with soap and water before touching the breast pump, milk collection parts, or your breast.
- Never leave the product unattended when the product is connected to the power outlet. Never use the product while sleeping or feeling drowsy.

- Risk of nipple trauma, tissue damage, or low milk expression. Never use a vacuum setting that causes pain or discomfort. Always turn off the breast pump if you experience pain. Break the seal between your breast and the shield with your finger before removing the product. Always contact your healthcare provider if milk expression is painful or milk output is low.
- Risk of milk contamination due to incorrect storage.
 - Always store and freeze breast milk in clean, plastic food-grade milk storage bags or bottles. Never store or freeze breast milk in the breast pump milk container.
 - Never use breast milk from damaged bags or bottles.
 - Never store the breast milk longer than the recommended maximum storage duration.

Choking hazard:

- Do not let children play with packaging material. Packaging materials can lead to choking and suffocation. Always keep the packaging material out of reach of children.
- Always keep small parts out of reach of toddlers and supervise children during use.

Electrical hazard:

- Do not expose the product to water, rain, moisture, or high humidity.
- Do not use the product when it is damp or wet.
- Never place or store the product where it can fall into water.
- Risk of electric shock or fire. Never immerse or run water over the motor. If the motor has been exposed to water or other liquids, never touch the product. If possible, unplug the product from the electrical outlet and contact the manufacturer.
- Disconnect the product from the power source before cleaning.

- Do not open the product, there are no user serviceable parts inside.
- Risk of short circuit. Never allow metal objects or fluids near the charging port or battery.
- Always unplug the product after charging and before cleaning.

Fire hazard:

- Do not expose the product to direct sunlight, naked flames or heat.
- Do not use the product near explosive or flammable materials.
- Do not dismantle, open or shred cells or batteries. Chemicals inside the batteries can combust upon contact with ambient air.
- Do not subject cells or batteries to mechanical shock.
- In the event of a battery leaking, do not allow the liquid to come in contact with the skin or eyes. If contact has been made, wash the affected area with copious amounts of water and seek medical advice.

Burn hazard:

- Risk of scalding. Never microwave or boil breast milk. Microwaving may create hot spots in the milk that can burn your baby's mouth. Always heat up breast milk by placing the container or bag under warm water (max. 37°C). Always check the milk temperature before feeding.

Tripping hazard:

- Always keep the power cable and the product away from usual walkways.

CAUTION!

Health hazard:

- Risk of pain or reduced product functionality. Never use the product while operating a moving vehicle.

- Risk of pain or reduced product functionality. Always check that the product is assembled correctly before use.

Electrical hazard:

- Rechargeable cells and batteries need to be charged before use. Always use the correct charger and refer to the manufacturer's instructions or equipment manual for proper charging instructions.
- Risk of electric shock or product damage. Only use an external adapter with the following specifications: input AC 100–240V, 50/60 Hz; output DC 5V 1.0A.

NOTICE

Risk of reduced milk quality:

- Never refreeze breast milk. Repeated freeze/thaw cycles destroy the structure of the milk.
- Never microwave or boil fresh or frozen breast milk. Microwaving or boiling may alter the composition of the breast milk.

Risk of damaging the product:

- Do not drop the product and avoid bumping.
- Always ensure all components are fully dry before use.
- Risk of battery damage. Never expose the product to excessive temperatures. Always charge the product in a cool, dry place between 5°C and 35°C.
- Risk of battery degradation. Always charge the battery before it is fully drained and before long-term storage.
- Risk of a damaged milk valve. Always handle the milk valve with care during cleaning. Replace the milk valve if damaged.
- Do not use aggressive chemical cleaning agents when cleaning the product.

- Risk of mineral residue on parts. Always use distilled water when boiling parts to prevent mineral build-up. Always wipe off any white residue and allow parts to air dry.
- Risk of damaged milk bags or bottles. Always handle frozen plastic bottles and milk bags with care. Frozen plastic bottles and milk bags may become brittle and break. Never fill the milk bags or bottles more than 3/4 full. Always leave space in the bag or bottle for the frozen milk to expand.

Disposal

The product is designated for separate collection at an appropriate collection point. Do not dispose of the product with household waste.

For more information, contact the retailer or the local authority responsible for the waste management.

Declaration of Conformity

We, Commaxx B.V. declare as manufacturer that the product BCB50 from our brand Alecto, produced in China, has been tested according to all relevant CE standards and regulations and that all tests have been passed successfully. This includes, but is not limited to the RED 2014/53/EU regulation.

The complete Declaration of Conformity (and the safety datasheet if applicable) can be found and downloaded via:

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